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PRODUCT DESCRIPTION

These wheelchairs have been designed for physically or mentally handicapped people with limited or no personal mobility only for transport.

This user's manual is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased.

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ASSEMBLY

- FOR REMOVAL

Push inward to release and turn the right footrest

Turn right and left foot to the left.

To remove the footrests from the chair, pull them upwards after turning them to release them from the rotating terminals.

- FOR ASSEMBLY

Place the footrests on the rotating pins by placing the right footrest on the right side and the left foot on the left side. Turn the footrests towards the center of the chair until they stop, so they will lock automatically into the chair.

Reconnect the calf strap to its stem.

FOOTRESTS 1

FOOTRESTS 2

INCREASE / REDUCTION OF FEET LIFTING (ON DEMAND)

- Lifting

Pull the release lever again until the foot is at the desired height.

- Lowering

Support the leg with one hand and release the release lever down with the other hand.

FOLDING HAND SUPPORT

• FOR RELEASE

Press the release pin and pull the arm up.

• FOR ASSEMBLY

Push the release pin down and gently push the arm into its anchor points. Then release the release pin

ATTENTION !!!

- Lock the armrests by securing the release latch mechanism before using the wheelchair.
- Never lift the wheelchair from armrests or footrests as these parts are detachable and can cause seat injury and / or injury to the user.

WHEEL LOCK

- If the wheels are pneumatic, check for adequate pressure.
- Move the wheel lock to the wheel if more braking action is required and remove the wheel from the wheel for less braking action.
- Tighten the lock nut securely with the same.
- Check that the wheel is rotating freely when the lock is not applied but is completely blocked when the lock is engaged.

ATTENTION !!!

- Wheel locks are not intended to brake your wheelchair. It's just there to make sure your wheelchair does not start to roll in.
- When stopping on uneven ground, always use wheel locks to prevent such rolling.
- Always use both wheel locks. Otherwise, your wheelchair could be overturned.
- Wheel locks are not designed to be used as brakes for a moving wheelchair.

RELEASE / INSTALLATION BACK WHEELS

1. Do one of the following:

Unscrew the end of the shaft and pull it together with the wheel out of the shaft assembly plate.

Pull the four-arm release shaft handle and pull the shaft and the wheel out of the shaft assembly plate.

2. Repeat STEP 1 for the opposite rear wheel.

3. To replace the rear wheel on the shaft support plate, reverse STEPS 1-2.

ADJUSTABLE HEIGHT

Close the armrest by pushing the release lever to the DOWN (CLOSE) position when the desired handle height is reached.

MAINTENANCE

In addition to maintenance and inspection, both the frame and the wallpaper can be thoroughly cleaned.

- To maintain its appearance, the wallpaper must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- Do not place the wallpaper in a tumble dryer. If the dirt is deep, the surface should be cleaned with a soft brush.
- The frame should be cleaned regularly with a mild detergent.

SAFETY ADVICE

What to check before use

- Quick-release axles (if any) on the rear wheels.
- Tires, tire pressures and wheel locks. The wheel locks will not work properly if the pressure is too low or the gap between the tire and the wheel locks is too long.
- The maximum user weight is 120 kg. This wheelchair should only be used to carry one person each time on the seat. Any other use beyond this limit violates the security regulations.
- Users should not sit in the wheelchair while driving on any means of transport.
- When vehicles are in motion, unoccupied wheelchairs should be secured by appropriate means. It is likely that traps or uneven terrain could lead the wheelchair to skip, especially when driving uphill or downhill.
- When driving in a step or tilting forward, the body must lean forward.

PAY ATTENTION TO THE FOLLOWING

- The construction and construction of this wheelchair are designed to provide maximum safety.
- The international safety standards currently in force have either been fulfilled or exceeded. However, users may be at risk by using their wheelchairs unsuitable.
- For your own safety, the following rules must be observed.
- Nonspecific or incorrect adjustment or adjustment work can increase the risk of accidents.
- Always wear light-colored clothes when moving in dark places so that you can look more easily from others.
- Be careful during your first course in this wheelchair.
- Know your wheelchair.
- Make use of your wheelchair on level ground as well as uphill and downhill.
- Do not attempt to reach objects if you need to lift them from the floor reaching between your knees.
- Do not attempt to reach objects if you need to move on to the seat.
- Do not use on roads, roads or motorways.
- Do not attempt to climb or descend with water, ice or oil membrane.
- Do not use unauthorized parts, accessories or adapters other than those approved by Mobiak.
- Do not stand in the wheelchair.
- Do not use the sideboard as a platform when entering or exiting the wheelchair.

While driving with your wheelchair, do not forget !!

Always use lifts and ramps. If these are not available, you can reach your destination with the help of two escorts. They must catch the wheelchair in securely seated areas. A wheelchair should never be lifted by a passenger. It should be pushed only.

IMPORTANT

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

WEEKLY

- Check the wheel pressure by following the instructions on the wheels.
- Check that the central bolt of the transverse arm is tight and in good condition.
- Clean the wallpaper with soapy water or sufficient products.
- Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
- Check the wheel bolts and adjust if necessary.
- Check for damage to front and rear wheels.

QUARTERLY

- Check and adjust if necessary: wheel forks, pulling wheels, propellers.
- Grease or oil all components that are subject to friction or impending wear.
- Tighten the screws and nuts.
- General revision of the chair's status and repair to whatever is necessary.

ANNUALLY

We recommend a full review once a year to a qualified service provider.

Protecting the environment

Should you find one day that your appliance needs to be replaced or if it is of no further use to you, think of the protection of the environment:

1) Do not dispose your appliance along with the rest of the public waste (this is also the meaning of the shown recycling sign).

2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your appliances must be disposed.

3) Correct disposal of your appliance helps the protection of the environment as well as the recycling of the appliance components.

4) The electrical appliances due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards.

WARRANTY:

Your wheelchair is guaranteed for **(2) YEARS**. No service or spare part is covered unless it has been inspected by an authorized service or the importer / manufacturer of the product. The warranty covers factory failures & NO damage from misuse - maintenance of the carriage.

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MOBIAK S.A