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Thank you for purchasing our products, hoping you will be satisfied with our products. This operation manual contains function, operation steps, attention, basic trouble solution and so on. To ensure your efficient use of the machine, please have a close read of this operation manual before operating it. Maybe, there are some pictures, which are different from what you have seen in the real model.

SAFETY NOTICE

- This product cannot be used for life maintaining, it is suggested that if any patient who needs oxygen treatment, please follow doctor's advice to choose the right flow and period for oxygen before using the machine.
- If any adverse reaction appeared or revealed during taking oxygen, please contact with equipment supplier or doctor as soon as possible.
- For serious patients, set an indicating device additional, any adverse reaction appeared, please contact with equipment supplier or doctor as soon as possible.
- Do not put nasal tube under bed cover or cushion, the oxygen that caused by machine turning on without breathing may be combustion-supporting.
- Turn off the switch if nobody takes oxygen.
- For oxygen can be combustion-supporting, keep oxygen concentrator far away from naked light or fire resource, no smoking or naked light around the patient.
- Before cleaning the dust on the net of oxygen concentrator, the plug must be pulled out in case electric attack.
- In using the machine, do not open the front and back cover at all. In case there are quality problem, do not dismantle it secretly. Any alarm or other abnormal phenomenon has been found, connect with equipment supplier or factor.

ATTENTION

The oxygen concentrator should be set to use in an environment without dust, corruption and toxicological harm gas. Air intake of the oxygen concentrator should be located in well-ventilated space, in case there are polluted air or smog in the oxygen. Ensure the bottom exhaustion smooth during operating, or else the machine will be over-heated. There is intermittent exhaustion sound during operating (10 seconds in intermission). 5 minutes are needed from oxygen concentrator from warming up to reach regular function.

- The machine is only for medical oxygen supply, and the oxygen concentration will be up to 90% when air outlet reaches its nominal flow.
- Humidifier shall adopt distilled water or cold boiled water, added shall be kept under the scale line.
- Use the humidifier brought with the machine, do not replace it at will, or else may cause patient uncomfortable or other harms.
- In case state indicator shows abnormal oxygen operator should declare to dealer or factory in favor of maintain.
- The humidifier, filter cotton and filter are the items needed to clean, among which humidifier should be cleared every 3 days, and external filter cotton should be cleared every 100 hours, and internal filter should be cleared every 3000 hours.
- Once open the adjust knob for flow in full, but flow meter shows zero, turn off the machine immediately and have a check for trouble.
- Do not turn on and off frequently: To restart the machine after turning off should have an interval no less than 5 minutes (namely, exhaust internal gas of the machine completely, for if air compressor turns on with pressure will shorten its life)
- Do not turn on the power with switch for flow adjusting shut up.

- Refresh the water in the humidifier every 2-3 days, especially in summers. If do not use it in several days, please pour out the water completely, and wipe dry the bottle

- Use the oxygen tube and humidifier brought with the machine or those of the same model, if change to use other model devices, please ensure close connection with the oxygen generator. The nasal cannula is only for the patient, and do not junk it at will.
- The oxygen tube, oxygen mask and nasal cannula that have touched with the patient should keep clean, disinfected and sterilized.
- The oxygen tube that have touched with the patient after each operation should be disinfected by wiping it with 75% medical use alcohol or other disinfecting methods. To prevent cross infection, do not share oxygen tube. Series products of this brand medical oxygen generator model V adopt pressure swing adsorption principle, which can separate oxygen, nitrogen and other gas from the air, at constant temperature, as soon as power is connected, the oxygen that meets medical use standards can be separated from air constantly. Oxygen is generated by pure physical method. The generator can supply 1-2 patients simultaneously, with steady oxygen flowing out, safe and reliable, low cost, adjustable flow. The key parts of the generator adopt anti-tiring and anti-aging design. There is no influence on indoor oxygen percent during the generator operating.

USING CONDITION

1. Ambient temperature: 10°C - 40°C
2. Relative humidity: 30%-85%
3. Air pressure: 700 hPa-1060 hPa
4. No corrosive gas and strong magnetic field around.


SCOPE OF APPLICATION

This device is mainly used for generating medical oxygen.

TECHNICAL PARAMETER

Model	MOBIAKCARE - Golden Series – Irene
Power Consumption(W)	480 Watt IRENE / IRENE NEB - 735 Watt IRENE 10Lt
Working voltage(V/Hz)	230 +/- 10% / 50 +/- 1
Flow Rate (L/min)	1-5, 1-10 ONLY FOR IRENE 10L
Concentration (%)	93 +/- 3
Outlet pressure (Mpa)	0,05- 0,08
Sound level (dB)	≤40 for Irene & Irene Neb, ≤51 Irene 10Lt
Electrical category:	Class II Type B
Product category:	Class IIb rule 11
Net Weight (Kg)	24 Kg Irene 5L & Irene Neb, 32Kg Irene 10Lt
Dimension (mm)	390x337x620
Oxygen percent alarm	When Oxygen percent is lower than 82%

OPERATION INTRODUCTION

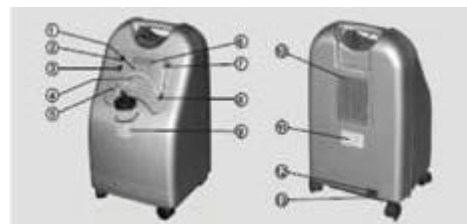Indicator light

Total 3 indicator lights, and the code of them are 1,2,3 from right to left, and their indication for each model are as follows:

1. Power indicator (green) Power-off alarm (red)
2. Trouble indicator, Working good (off), Defective happened (Red)
3. Oxygen percent (green-normal, red alarm the Oxygen percent is lower than 82%)
- 2) Resettable Short circuit protection switch
- 3) Power switch
- 4) Function switch for setting timing of oxygen supply
- 5) Atomization outlet (for atomization type only)
- 6) Liquid crystal display It can display total operation time and start time of this time, which May guide the user to operate generator in a more Scientific and safe way.
- 7) Knob of oxygen flow meter switch

The other name of knob of oxygen flow meter Switch is flow control valve.
It adjusts and controls the outlet oxygen flow.

7. If the patient need timing oxygen treatment, first, press the M switch of continue / timing, then the timing clock flickers, and adjust the timing time by keys, ▲ ▼ and finally, press ← confirm the timing state. Under the timing state, it is able to switch to continue state by pressing the M switch which can covert between continue / timing.

5. Insert the intake end of nasal cannula onto the outlet of Humidifier, then set the nasal cannula over patient's ears, insert the nasal tube into patient's nostrils to for starting the oxygen treatment. The oxygen concentrator has a firetrap connects it following to the photo (A). It is concealed to use Water traps (accessory NOT included, can be acquired separately), especially during winter period, to avoid the formation of water in the tubes due to temperature difference. Case you need to connect the Firetrap with the Water trap, follow photo (B). If needed use a small piece of tube to connect them.

6. When finished the treatment, turn off the power, if there is discontinuity use, please unplug the power plug.

fig. A

fig. B

8. Firstly, Power-On-Self-Test, power on the concentrator, then check the color of the indicator on the front panel.

Green - The concentrator is in a good working status.

Red - The concentrator may need to do maintenance or some parts were damaged, the oxygen purity level was dropped to a dangerous level..

9. In case oxygen generator is connect with power but the whole machine is still in the state of power off with alarm sound, please check out the connection part of power whether it is in good connection, or whether there is a power off in external power supply. **ATTENTION IMPORTANT** If the Oxygen Concentrator is used for constant use of 24 hours / 7 day per week, the user is obliged to give to the device a rest for 15 minutes every 8 hours.

MAINTENANCE DESCRIPTION

1. In the condition of power off, make a clean for the outside body by soft towel with little detergent, and then wipe it up with dry towel, once or twice per month.

2.It is a critical step for daily maintenance to clean filter net for gas intake, at least twice a month.

fig. 1

fig. 2

fig.3

Detail steps:

Please clean the filter window by liquid detergent, then rinse the filter window by clean water, please clean the black foam filter (fig.1) every 3-4 days with water & wait the filter window dry in air. Replace the black foam filter every 3000 hours.

3.Clean air intake filter.

Replace air intake filter (fig.2) an interval about 1500 hours, open the filter window on the back casing, then unplug and replace the filter bottle inside. The disk shaped bacterial filter (fig.3) needs to be replaced by your home provider according to service & maintenance schedule. Usually every 1500 hours and incase of leasing to every patient have to replace with a new one.

Please follow above instructions & use ONLY spare parts approved & provided by MOBIAK S.A

Nebulizer System

To use the IRENE nebulizer system (only model "IRENE Neb"), shut the oxygen flow from the flow regulator (clockwise), place the nebulizer mask in the slot and rotate the movable point to terminate and start the air flow. After the end of treatment rotate inversely to shut the supply of air and restore the flow regulator to the desired oxygen flow.

SELF DIAGNOSES FUNCTION

Alarming code list: In alarming mode, the alarming indicator will be on as well as the sound indication, and the number was displayed on the screen will notify the end user for what happened.

Code 10000 - Overheat warning

Code 3000 - Overpressure warning

Code 4000 - Low pressure warning

CONDITION FOR TRANSPORTATION AND STORAGE

Environment temperature scale: -20 °C to 45 °C

Comparative humidity scale: ≤95% Air pressure scale: 500 - 1060 hpa

Note: Please do place the machine at a good ventilation area, please make sure that the exhaust port underneath the machine not be covered.

MEDICAL EQUIPMENT - HOME CARE

MOBIAK A.E: Μ.ΜΠΟΤΣΑΡΗ 96-98, ΧΑΝΙΑ-ΚΡΗΤΗΣ, 28210-63222
MOBIAK S.A: 96-98 Μ. Μπotsari STR. Chania, Crete, Greece, +30 28210-63222
CE0653

www.mobiakcare.gr